
ENGLISH

Nasal-Aire® II Critical Care

Hospital Applications **Single-Patient Use** **Disposable**
 Nasal-Aire® II Critical Care ventilator interface is intended for use with a CPAP or Bi-Level device to provide noninvasive ventilation in adult patients for the treatment of respiratory insufficiencies or obstructive sleep apnea. This single use disposable interface is intended for hospital settings.
 *Note: The Nasal-Aire® II Critical Care has a 22mm female connector for use with a 22mm male hose. When used with a 22mm female hose, a male-male adapter is required.

System Components
 Nasal-Aire® II Critical Care consists of a soft nosepiece with exhaust ports and two soft silicone nasal cannulas for fitting into the nasal openings. Connected on each side of the nosepiece is a length of tubing. The two lengths of tubing are joined at the opposite ends by a "Y"-shaped coupling, forming a loop with the nosepiece at one end and the "Y" coupling at the other. At the base of the "Y" coupling is a port for connection of the Nasal-Aire® II Critical Care to a positive pressure ventilation device.

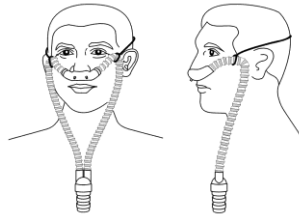
Preparation for Use
 1. Remove Nasal-Aire® II Critical Care from package.
 2. Complete circuit assembly before use.

Directions for Use
 1. Position the cannula facing the nose, and position tubing with "Y" connector resting on chest.
 2. Grasp nosepiece with thumbs and fingers under each nasal insert and position near nostrils.
 3. Using thumbs, press inserts into nostrils until base of Nasal-Aire® II Critical Care nosepiece is flush against the nose.
 4. Turn ventilation device on.
 5. Attach "Y"-adapter to CPAP tubing.
 6. Once airflow starts, adjust Nasal-Aire® II Critical Care to fit for comfort and seal. Periodic adjustments may be necessary.

Warnings and Precautions
 ● Federal (USA) law restricts this device to use by or on the order of a physician.
 ● The Nasal-Aire® II Critical Care should not be used unless the ventilation system is turned on and operating. The exhalation ports should never be blocked.
EXPLANATION OF WARNING: CPAP systems are intended to be used with a mask or special nasal cannulas with intentional ports to allow a continuous flow of air through the breathing circuit. When the CPAP machine is turned on and functioning properly, fresh air from the CPAP machine flushes the exhaled air through the intentional leak port. However, when the CPAP machine is not operating, fresh air will not be provided through the circuit and exhaled air may be rebreathed. Rebreathing of exhaled air can, in some cases, lead to suffocation.
 ● Use Nasal-Aire® II Critical Care only with a positive pressure ventilation device.
 ● Nasal-Aire® II Critical Care is intended for ventilators that do not incorporate, use, or require an exhalation valve.
 ● Nasal-Aire® II Critical Care is intended for single-patient use only. Do not sterilize or use disinfectants on this device.
 ● Nasal-Aire® II Critical Care may deactivate the pressure alarm on some ventilation systems. Confirm that your ventilation system pressure alarm works with the product before using the pressure alarm.
 ● Some users may experience skin redness. If this occurs, consult your doctor.
 ● Do not expose this device to extremes of temperature. Store the Nasal-Aire® II Critical Care at temperatures between 0°C (32°F) and 54°C (130°F).

Pressure Versus Leak Rate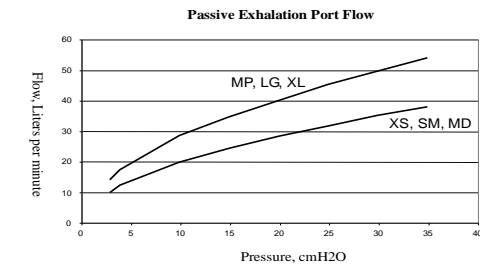

Deadspace

Size	Cannula Deadspace (CC)
XS	8
SM	8
MD	13
MP	14
LG	17
XL	18

Reorder Information

Extra Small Nasal-Aire® II Critical Care CXS 1
 Small Nasal-Aire® II Critical Care CSM 2
 Medium Nasal-Aire® II Critical Care CMD 3
 Medium Plus Nasal-Aire® II Critical Care CMP 4
 Large Nasal-Aire® II Critical Care CLG 5
 Extra Large Nasal-Aire® II Critical Care CXL 6

NOTE: Retain these instructions and the lot number of your Nasal-Aire® II Critical Care. Please reference the lot number if you need to contact Salter Labs or your distributor about this product.

Manufactured for: Salter Labs
 30 Spur Dr., El Paso, TX 79906 USA
 Tel: (800)-421-0024 www.salterlabs.com

Manufactured by: Modern Medical Equipment Mfg. Ltd.,
 5fl, 35 Tai Lin Pai Rd., Kwai Chung, Hong Kong
 Tel: +852 2420 9068 Fax: +852 2481 1234

Covered by Patents: US 6595215, US 6807967, US 6863069, US 6994089, US 6997177, US 7059328, US 7188624, AU 781624, AU 782824, EP 1317941. Other patents pending.

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Produceret af: Modern Medical Equipment Mfg. Ltd.,