<b>EN</b>
<b>DEVICE DESCRIPTION:</b> The device is non-sterile, disposable and for single-patient use. Oxygen tubing is a conduit to delivery oxygen from the source gas to the patient. <b>INTENDED PURPOSE:</b> Oxygen tubing is a conduit to delivery oxygen from the source gas to the patient. <b>INDICATIONS:</b> To extend the length of the oxygen tubing <b>ENVIRONMENT:</b> Hospitals, sub-acute, medical clinics, pre-hospital, home, surgical centers, skilled nursing facilities <b>PATIENT TARGET GROUP:</b> Infant, pediatric, adult <b>EXPECTED CLINICAL BENEFITS:</b> Lengthens tubing, allowing patient increased mobility <b>CONTRAINDICATIONS:</b> None Known <b>WARNINGS:</b> If oxygen is in use, do not use near flame or heat source. Position tubing to avoid strangulation. Patient may become hypoxic if oxygen flow is interrupted. Do not use with low or high flow nasal cannulas. <b>CAUTIONS:</b> Consult physician if patient develops infection, skin irritation or material sensitization. Follow cleaning instructions to avoid risk of infection and contamination. Do not place anything on supply tubing that may obstruct flow. Keep excess tubing loosely coiled and out of the way to prevent kinking and tripping hazard. <b>INSTRUCTIONS:</b> Contact your country’s Competent Authority to report any serious incident. Dispose of device in accordance with local, state or national regulations. Follow instructions for proper care of device to prevent risk of contamination and injury. This product is disposable and is not intended for prolonged use. Check for leaks at connections and for proper placement prior to use. Discard and replace if the cannula or tubing becomes soiled, discolored or stiff. Use water trap if excess condensation is present. <b>FOR NEBULIZERS, NASAL CANNULAS, OXYGEN TUBING, ETCO2 TUBING WITH OXYGEN, RESUSCITATION BAGS:</b> <b>WARNING:</b> To reduce the risk of misconnections and patient injury, always trace tubing from gas source to the medical device (e.g., nasal cannula, nebulizer) before connecting. <b>FOR ETCO2 TUBING WITHOUT O2, BLOOD PRESSURE CUFFS, CATHETERS, SUNSTIM ANY OTHER PRODUCTS THAT HAVE TUBING ATTACHED TO THEM:</b> <b>WARNING:</b> To reduce the risk of misconnection and patient injury, always trace tubing and lines back to their point of origin to verify that correct connections are made.

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