



USE, PACKAGING AND SHIPPING INSTRUCTIONS




PROGRAM COMPONENTS

- 2 to 3 SunCycle 12 gallon recycling container with SunCycle label and absorbent pads in bottom
- 4-millimeter biohazard-labeled plastic bags for each recycling container
- Packaging tape and tie wraps
- Inner brown box with pre-labeled, prepaid return shipping label (DO NOT THROW AWAY)
- 4-millimeter large biohazard-labeled plastic bag for lining brown shipping box

The SunCycle™ Program contains everything needed to properly and safely package and dispose of your SunMed® disposable laryngoscope components. When properly packaged by the waste generator, this mail-back system meets all FedEx requirements for sending regulated medical waste through the U.S. mail to our disposal site. Do NOT place this system in FedEx pick up if it is not properly and securely assembled.



SunCycle is intended for recycling of SunMed single use laryngoscope blades and handles. Other medical waste cannot be accepted. Do not place sharps, hazardous or radioactive-contaminated waste in this container system.

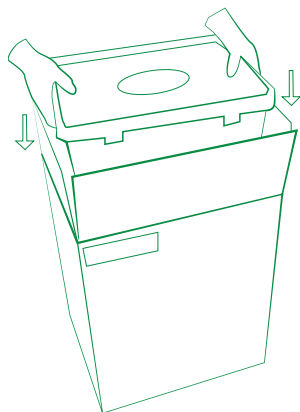

PREPARING FOR USE

1. Open the shipping box. Inside is a second brown box REMOVE this inner box from the shipping box and SAVE. This inner box is pre-labeled for return once the SunCycle™ containers are full.
2. Remove the white SunCycle containers and place in appropriate locations for disposable laryngoscope collection.
3. Your SunCycle container is now ready for use.

USING SUNCYCLE™

1. Place the SunCycle container(s) near the point of use.
Keep the container(s) away from children and untrained persons.
TO PREVENT INJURY, DO NOT OVERFILL THE RECYCLING CONTAINER.
2. Following use of any SunMed disposable fiber optic laryngoscope blade and/or handle, dispose item(s) in the SunCycle container.
3. Once the contents reach the maximum fill line shown on the SunCycle container, firmly secure the round sealing cap provided onto the container. Follow instructions for closure as directed on container(s) to ensure a tight and secure seal. A snapping sound indicates the lid cap is seated properly.

PREPARING RECEPTACLES FOR RETURN

1. Place the receptacle into the brown inner box.
2. Close the brown box. Next, fold the large flaps down so that the edge of each large flap meets at the center. Use one strip of the tape provided to securely seal the entire length of the flap surface where the edges meet, allowing the tape to overlap down the sides of the brown inner box. If the tape does not appear to be secure, apply your own additional packaging tape to secure the flaps of the brown inner box.
3. Make sure your pre-paid FedEx label is on the outside of the box – There is NO CHARGE to your facility for this shipment.

MAILING

1. Enter your complete return address on the merchandise return service mailing label located on the top of the brown FedEx shipping box. NOTE: P.O. boxes are NOT allowed as a valid return address.
2. Schedule a FREE FedEx pickup online by visiting www.FedEx.com or take to a FedEx drop-off location.

SunCycle is a SunMed program
in partnership with



FOR MORE INFORMATION, CONTACT SUNMED

1-800-433-2797 | info@SunOneSystem.com
www.SunOneSystem.com/go-green/

IFU-10000 Rev.1